Made in U.S.A. by  3M Health Care,  
2510 Conway Ave. St. Paul, MN 55144  
(U.S.A.) 1-800-228-3957 Visit our website [www.littmann.com](http://www.littmann.com)

Not made  
with natural  
rubber latex.

Consult  
instructions  
for use.

Littmann®  
Cardiology IV™  
Stethoscope

3M, Littmann, the L Littmann logo, the L,  
and Cardiology IV are trademarks of 3M.

Used under license in Canada.

© 2015, 3M. All rights reserved.

Issue Date: 2015-05

34-8717-5283-7

**EC REP** 3M Deutschland GmbH  
Health Care Business  
Carl-Schurz-Str. 1  
41453 Neuss  
Germany

**3M**

# 3M™ Littmann® Cardiology IV™ Stethoscope User Manual


## Explanation of Symbols

 Consult instructions for use.


 Not made with natural rubber latex.

## Instructions for Use


Insert earlips in a forward direction.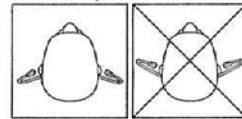

## Adjust the Headset Tension

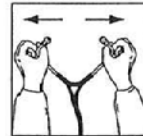To reduce the spring tension, pull the eartubes apart.

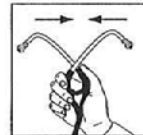To increase the spring tension, squeeze the eartubes together.

## Select the Active Side of the Chestpiece

Only one side of the chestpiece is acoustically active at a time, identified by a mark on the stem. Rotate the chestpiece to change which side is active.

## Listen to Low and High Frequency Sounds with Littmann Tunable Diaphragms on Each Side of the Chestpiece

Use the large side for adults and the small side for pediatric or thin patients, around bandages and carotid assessment.

*Low frequencies:* apply light pressure to the chestpiece

*High frequencies:* apply firm pressure to the chestpiece

## Convert to a Traditional Open Bell

Pinch the rim of the diaphragm on the small side; pull and lift from the chestpiece.

Attach the sleeve by placing one side on the edge and stretching it over the chestpiece.

To install diaphragm, remove the nonchill sleeve. Insert the edge of the diaphragm into the groove and roll the rim around the chestpiece.

GB

### ► Cleaning, Disinfection and Storage

Clean your stethoscope between exams using a 70% isopropyl alcohol wipe or a disposable wipe with soap and water. A 2% bleach solution may be used to disinfect your stethoscope; however, the tubing may experience a slight discoloration after exposure to bleach. The tunable diaphragms, non-chill bell sleeve and eartips may be removed for thorough cleaning. To remove the eartips from the headset, pull firmly. To assemble the eartips, push the small side of the eartip firmly onto the eartube until it snaps fully into place. Ensure all parts and surfaces are dry before reassembly. Be sure the eartips are firmly attached to the headset before inserting them your ears.

Do not immerse your stethoscope in any liquid.  
Do not subject your stethoscope to any sterilization process.

Avoid storing your stethoscope in extreme heat.  
To prevent staining of stethoscope tubing, avoid contact with pens, markers, newsprint, or other printed materials.

### Littmann Stethoscope Service and Warranty Program

Please register your stethoscope online at: <http://www.littmann.com/warranty-registration>.

The Littmann Cardiology IV stethoscope is warranted against any defects in material and workmanship for a period of seven (7) years. Within the warranty period, repairs will be made without charge upon the return of the stethoscope to 3M, except in cases of obvious abuse or accidental damage.

For service and repair in the U.S.A., please visit [www.littmann.com/service](http://www.littmann.com/service) or call 1-800-292-6298. If you are outside of the U.S.A., please visit [www.littmann.com](http://www.littmann.com) for the contact information of your local 3M office.

For more information, visit [www.littmann.com](http://www.littmann.com).

3M, Littmann, the L Littmann logo, the L, and Cardiology IV are trademarks of 3M.

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